

Round Marking Bur



RMB

Externally Cooled Twist Drill



ECD 2.0
ECD 3.0
ECD 3.3
ECD 4.0
ECD 4.3

Pilot Drill



PD 2-3
PD 3

Countersink



CS

Bone Tap



*TAP 3.75
*TAP 4.0

Implant

Drilling Sequence

3.3 mm Self-Tapping/Acid Etched	RMB	2.0 mm ECD	2-2.75mm PD	2.75 mm ECD	3.4 mm CS	*3.3 mm ECD	*3.3 mm TAP
3.75 mm Self-Tapping/Acid Etched	RMB	2.0 mm ECD	2-3mm PD	3.0 mm ECD	4.1 mm CS	*3.3 mm ECD	*3.75 mm TAP
4.0 mm Self-Tapping/Acid Etched	RMB	2.0 mm ECD	2-3mm PD	3.0 mm ECD	4.1 mm CS	*3.3 mm ECD	*4.0 mm TAP
5.0 mm Self-Tapping/Acid Etched	RMB	2.0 mm ECD	2-3mm PD	3.0 mm ECD	3-4.0mm PD	4.0 mm ECD	*4.3 mm ECD *5.0 mm TAP

*optional for dense bone

If you are placing Acid Etched implants, be sure to use the Acid Etched bone tap which has been specifically designed for the double thread design on the Acid Etched implants.