



Rubber Sep #812045 & 900675

Block-out, Separating, & Protection Agent

Instructions for use:

- 1. Mix thoroughly and thin with water to desired consistency. It is advisable to transfer a small amount of *Rubber Sep* into a separate container to prevent evaporation of moisture from the entire bottle.**
- 2. Apply with an instrument rather than with a brush.**
- 3. Layer *Rubber Sep* to obtain proper thickness and maintain controllable drying times.**
- 4. *Rubber Sep Pink* turns dark red when it is dry. *Clear Rubber Sep* dries clear. Air flow or heat will hasten drying times.**

Note: *Rubber Sep* is not intended for intraoral use and may cause nausea. It has a strong odor in large amounts. Use with adequate ventilation. *Rubber Sep* is not harmful to the skin but avoid contact with clothing.



**23 Frank Mossberg
Drive, P.O. Box 2967,
Attleboro, MA 02703
508-226-5660**

800-243-9942

Fax 800-531-2685

www.sterngold.com