

# EZ PickUp™

*ERA PickUp® has a new name:  
Same proven & trusted formula; EZ new name.*

**Tissue-colored, self-curing attachment processing material for any attachment**

## Advantages

- Odorless & Tasteless.
- Low curing temperature over traditional acrylics.
- Less than 1 percent shrinkage resulting in a more accurate and complete pickup of the attachment.
- EZ PickUp™ material will not irritate freshly sutured tissue like methyl methacrylate will.
- Can be used to pick up any attachment.

**CALL NOW  
TO ORDER**



[www.sterngold.com](http://www.sterngold.com)

**800-243-9942 • 508-226-5660**

**EZ PickUp™ Syringe and Tips**  
1 syringe @ 15 grams and 15 tips  
**EZ PickUp™ Tips**  
15 tips  
**SternVantage Varnish LC**  
30 ml bottle  
**EZ PickUp™ Syringe, Tips  
& SternVantage Varnish LC**  
1 syringe @ 15 grams, 15 tips and  
1 bottle of 5 ml Varnish LC

**Item #**  
220235  
  
220233  
  
221001  
  
220237

**EZ PickUp™**  
**Voted Best**  
*Pick Up Material*



*EZ PickUp™ will chemically bond to acrylic only if a primer is used. Use SternVantage Varnish LC as a primer for acrylic.*

# Sterngold's Prosthetic Procedure

Utilizing:

SternVantage® Varnish,

EZ PickUp™ &


QuickLine™



**1** Use SternVantage® Varnish to prime the two recesses over the metal jackets and light cure.



**2** Add acrylic or composite over the top and sides of the metal jackets. Sterngold's EZ PickUp™ material is suggested.

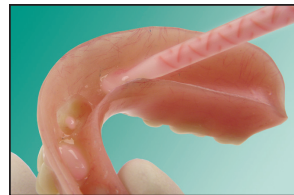

**3** Place additional resin in the recesses of the overdenture and seat the prosthesis in the mouth.



**4** Passive seating is most important. If the tissue is displaced, it will make accurate seating of the attachments very difficult.



**5** Remove the denture. Fill any defects with resin and finish the prosthesis. Excess EZ PickUp™ can easily be removed from the unvarnished areas.



**6** A soft reline material, like Sterngold's QuickLine™, is recommended to cushion the tissue and implants during the healing phase.